


Face Shield Assembly

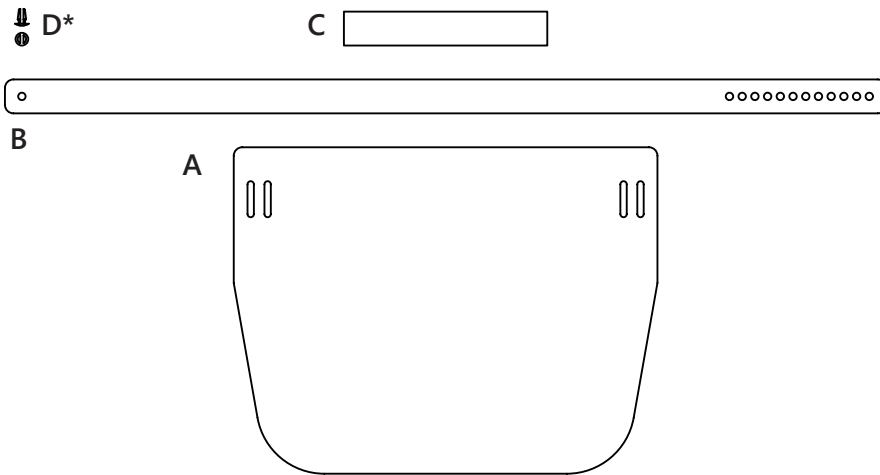


Each face shield comes with four components: the plastic shield (A), head strap (B), foam (C), and clip (D).

**the side and top view of the clip is shown.*


Assembly Instructions:

1. Remove the protective film on the shield and head strap.
2. Thread the head strap through the slots in the shield.
3. Remove the protective film on the foam piece, and place the adhesive side in the middle of the head strap, on the side that is not against the shield.
4. Insert the clip through the isolated hole in the head strap.
5. Hold the shield in front of your face, and wrap the strap around your head. Insert the clip into a second hole so that the shield fits comfortably on your head.
6. To remove the clip, squeeze the end of the clip and pull.
7. Slide the shield forward or backward to the desired distance from your face.


Head Strap with Foam

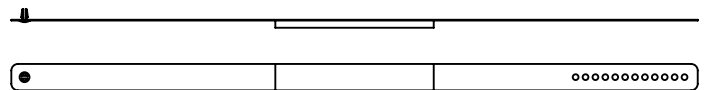

Front View, Assembled

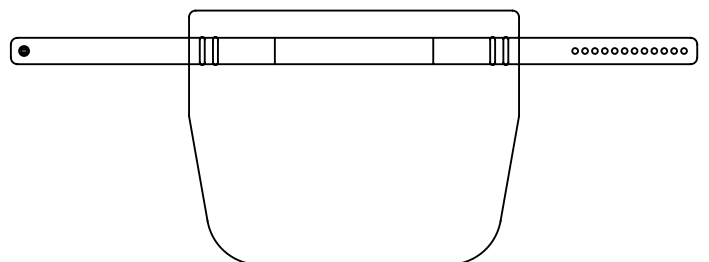

Side View, Assembled

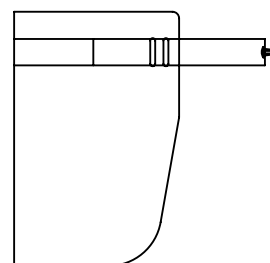

Thank you for your order! Need more shields?
Visit cooperenterprises.com or call us at (419) 347-5232.

